





3.5 cWHUSODFHUDDGDUDH□




IWL00EDNDPDGUDHRYDSnINGGHW□  
 95116INGGHWPNWHDOOWLGSODFHUDDPHG□  
 GHWNMWDGHINGGUIFNHDYIGDERUWIUn□  
 IDYHO□

4. □ 19b1111□

4.1 UOILDYINGGHW

6INGGHWNDIWOIDIWDDWWHDYIGDUH□  
 EIWWUHNGGPRWWUOOLDYIH□

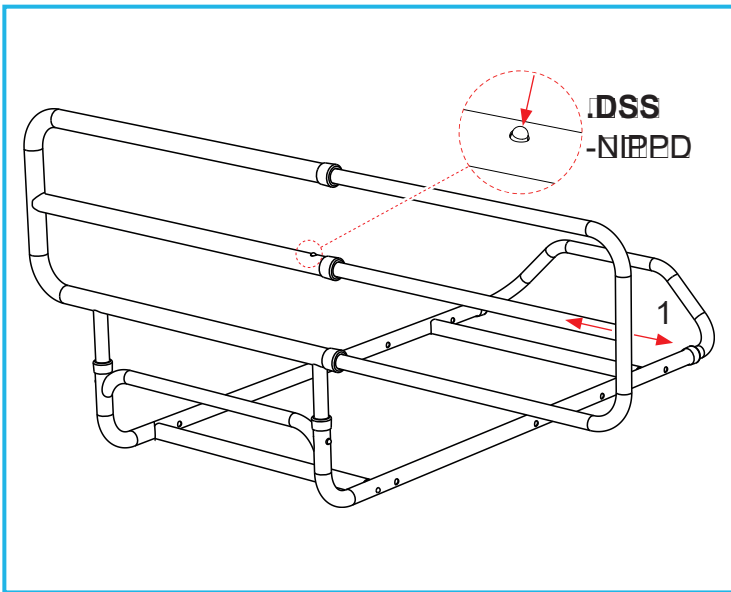

UDWWWIQDVIQVNHWDWVIQVNHWW□  
 IWQQLQVNHQDIDnQKMNMIENHWWLM□  
 VIQVNHWWIWQQLQVNHQDVNQDSSNIPP□  
 NLENDUSnSDWV□


26 : 6HWLODDWWNDSSNOIPPDKDUIIWKHOWLKHOTW□

4.2 LNDINGGHWOIG

6INGGHWIWOILNHDPnWHYDUDWIG□  
 IUDWwOnWDDYIGDUHnUIHIDWwWID□  
 INGGHWIWOILNHDWUFENRFKKnOOL□  
 NDSSNOIPPDRFKNMWHGDLINGGHW□  
 IWOILLKYGNNGGUIFNH□

4.3 7DERUWINGGHW

6INGGUIFNHWNDWDERUWnDWWDYIGDUH□  
 NDEIGGDIH□

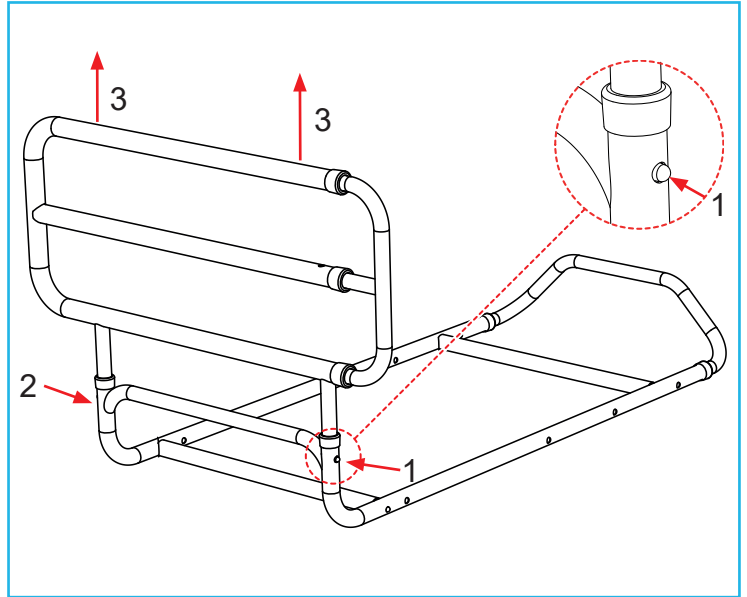

7UENREKKnNDUHQQUDVIQUIFNHWV□  
 NQDSSNIPPDRFKUDVHDQCLMIQVNV□  
 UIFNHWWLMNQDSSNIPPDRVVDQDUIDnQKnH□  
 8SSUHSDVWHIWIQVWHUMLD□  
 6HWLDWWEhDNQDSSNIPPBUQDKDURVVDW□  
 IDnQKnHQQUDVHDQWMIQVNMUIFNHW□  
 IDnQEDVUDPHQ□  
 UDWWnWHUPRQWHUDVIQUIFNHWMHDVQLWV□

4.4 7DDYHK6NGGWHW

6INGGHWOHYHUHUDPHGIWNGGHWPRWHUDW□  
 SnINGGUIFNHWIUDWwWDERUWIUHUH□  
 UIOMDGH□



RDNDUGERUUHEDGHWFKUDDQWVNHWIUnQ  
 VIQVNHW

4.5 Sätt tillbaka Mesh Skyddstyget

Sk t ätskyddet sä gskydds ä ke h  
 äst ed ka db e ästet  
 Se till att ätskyddet ä helt astsatt i a  
 a vä d i g

## 5. RENGÖRNING

### 5.1 Rengöring av sängskyddet

**OBS:** Please check the suitability of the cleaning products before use. Do not use abrasive cleaners or cloths as these could permanently damage the surface and invalidate any warranty.

1. Clean regularly using a general purpose, neutral detergent and warm water.
2. To disinfect, wipe with a dilute bleach solution, and rinse thoroughly with fresh water.
3. Allow product to fully dry prior to use/storage.

### 5.2 Cleaning the Mesh Cover

1. Hand-wash the mesh cover using a mild fabric detergent and warm water.
2. Rinse the mesh cover with warm water.
3. Allow the product to dry fully before use/storage.

## 6. MAINTENANCE

Check the bedguard is correctly positioned and that all button-clips have engaged securely.

Inspect the product for signs of wear or damage at regular intervals. Any suspected faults or damage must be reported to the supplier and the product taken out of use immediately.

## 7. ACCESSORIES & SPARE PARTS

**P42744:** Spare Mesh Cover (white).

For more details please visit our website:

[www.nrshealthcare.co.uk](http://www.nrshealthcare.co.uk) or call Customer Services.

## 8. PRODUCT DISPOSAL

This product should be thoroughly cleaned prior to disposal.

This product is manufactured from metal and plastic and can be disposed of through Local Authority Household Waste and Recycling Centres or through Local Trade Waste Collectors.

## 9. PRODUCT GUARANTEE

NRS Healthcare guarantee this product for a period of 12 months, from the date of purchase, against faulty materials and workmanship.

This guarantee does not cover; fair wear & tear, abuse, misuse, interference or tampering with any

part of the product.

Any guarantee issued by NRS Healthcare does not affect your statutory rights.

If you have any problems with our products, please contact us at the address below.

If you have experienced an injury, contact us but feel free to contact the Medical and Healthcare products Regulatory Agency at:

[devices.regulatory@mhra.gov.uk](mailto:devices.regulatory@mhra.gov.uk)

**UK  
CA** NRS HEALTHCARE Sherwood House,  
Cartwright Way, Forest Business Park, Bardon  
Hill, Coalville, Leicestershire, LE67 1UB, UK.

**CE** NRS HEALTHCARE 4C Altona Road,  
Blaris Industrial Estate, Lisburn, Co  
Antrim, Northern Ireland BT27 5QB

Tel: Customer Services: 0345 121 8111

Email: [customerservice@nrshealthcare.co.uk](mailto:customerservice@nrshealthcare.co.uk)

**NRS Healthcare** is a leading supplier of community equipment and daily living aids.  
Please visit one of our websites to view our full range of products:  
[www.healthcarepro.co.uk](http://www.healthcarepro.co.uk) or [www.completecareshop.co.uk](http://www.completecareshop.co.uk)